

PHOTO - DEMONSTRATION

Emergency Chair (Model 400) depicted in use. All chair models function similarly.

Page 1 of 2
(See Reverse)



1



The chair is easily accessible as it is wall mounted near each stairwell for use in emergencies.

4



Wheel evacuee into position for evacuation down stairs or relocation up stairs.

2



- Simply remove the chair from wall and unfold into open position.
- **Secure latch**, then chair is ready to use.
- Front handles can be easily removed by depressing quick release button for easier access.



5



When second person is in position and evacuee is strapped in, lift evacuee.

3



- Assist person into chair.
- Replace handle if one was removed while loading evacuee.
- Secure chest straps.
- Secure arm and leg straps if needed.



6



***Evacuate** down stairs in event of emergency evacuation out of structure.

*Evacuee should always be facing upstairs.

ARE YOU PREPARED?

PHOTO - DEMONSTRATION

Emergency Chair (Model 400) depicted in use. All chair models function similarly.

7



*Relocate up stairs in event of smoke, fire, debris or other obstruction by simply reversing path.

*Evacuee should always be facing upstairs.

8



Once out of the building evacuee can be wheeled quickly and safely out of harms way.

9



Lightweight, the Safety chair can be easily folded for re-entry into the building or can be re-mounted on wall for future use.

10



The Safety Chair's 6 points of lift allow additional people to assist in the evacuation if necessary. The side handles are also removable in the event of a tight stairwell in older buildings.



Questions?

Call Toll Free: **1.877.379.2638**

Monday - Friday 8 - 5 PST

Please visit our web-site for more information:

www.safetychairs.net



ARE YOU PREPARED?